WASH & CARE.

STEP 1

Wash the mask under running water or place it in a bowl with water. The water temperature doesn't effect the self-disinfection. Can be up to 90°C.

STEP 2

Stains and make-up residuals can be removed with a bit of mild hand soap. Rinse thoroughly to take out all soap before drying.

STEP 3

Air dry your mask. It dries fast close to a heater or using a hair dryer. Avoid excessive heat, e.g. open flame and really hot surfaces.

DOES & DON'T:

You can hand wash the mask up to 30x We recommend washing it once per week. This gives you > 6 months of usage.

Avoid detergents, softeners and bleach. The anionic surfactants can block the anti-microbial surface of the mask. Avoid washing machine and dryer. They can mechanically harm the mask.

HOW TO USE.

STEP 1

Wash your hands before putting on your mask.

STEP 2

Place the mask over mouth and nose. Put on the earbands.

STEP 3

Adjust the earbands and the nose wire with your index fingers for improved fit.

STEP 4

Do a quick test. Breathe easily, no air should escape through the sides.

STEP 5

Before taking off the mask, wash your hands.

STEP 6

You can now store the mask in the protective bag.

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